GENERIC ADRENACLICK[®] AUTO-INJECTOR SKILLS TEST

| Name School Place a (1) in the box if the skill is attained. | | |
|--|---|---|
| | | |
| 1 | Identify five signs and symptoms of anaphylaxis. | |
| 2 | Have someone call 911 (If no one available, administer Generic Adrenaclick® then call 911). | |
| 3 | Remove Generic Adrenaclick [®] from the case if ready to use. (Note: The Generic Adrenaclick [®] auto-injector is designed to go through clothing or directly on skin). | |
| 4 | Pull off GRAY end cap with the [1]. Be careful not to touch the RED tip. | |
| 5 | Pull off GRAY end cap with the [2]. | |
| 6 | Place the RED tip of the Epinephrine auto-injector against the middle of the outer thigh area (upper leg). | |
| 7 | Press the Generic Adrenaclick [®] firmly against the thigh until the needle enters the thigh. Hold in place while slowly counting to 10. | |
| 8 | Remove the Generic Adrenaclick® from the injection site and monitor until help arrives. | |
| 9 | Have a second dose readily available to repeat steps 4-8, if symptoms do not improve or worsen in 5 to 15 minutes. | |
| sc | CORE FOR NUMBER OF SKILLS CORRECTLY PERFORMED | 9 |

Potential Signs and Symptoms of Anaphylaxis

Mouth: Itchy, swelling of tongue and/or lips Throat: Itchy, tightness/closure, hoarseness, trouble breathing/swallowing Skin: Itchy, hives, redness, swelling, red watery eyes Gut: Nausea, vomiting, cramps, diarrhea

Store and Use

- Room temperature
- DO NOT refrigerate
- Do not keep in car during hot/cold weather
- Check expiration date, most expire within one year

Call 911 immediately after using the auto-injector.

Transport by emergency personnel and monitor for further medical assistance in the emergency room at least 4 hours.

Which Generic Adrenaclick® should you use?

- (0.3 mg) if > 66 lbs
- (0.15 mg) if 33 66 lbs

STAFF SIGNATURE

Heart: Pale or blue skin color, dizzy/faint, weak pulse **Neurological:** Sense of "impending doom,"

Lung: Short of breath, wheeze, repetitive cough

irritability, change in alertness, mood change, confusion **Other:** Itchy, red, watery eyes

- View color through window of unit: Should be clear. If brown, need new one
- Dispose of auto-injector properly